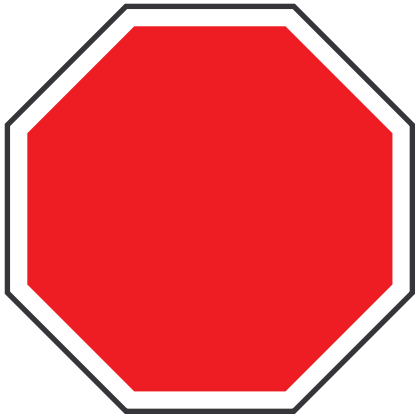
STOP: READ FIRST
ARRET: LISEZ D’ABORD
STOP: EERST LEZEN
HALT: BITTE ZUERST LESEN
STOP: LEGGERE PRIMA
PARADA: LEA ANTES

RECYCLING AND DISPOSAL

Cardboard may be recycled. Dispose of foam in accordance with state or local ordinances.

READ BEFORE UNPACKING

Apparent Loss or Damage

Upon receipt of this shipment, if any evidence of loss or damage is revealed through visual inspection, then it must be noted on the freight bill and signed by the carrier's agent. Failure to do this may result in the carrier's refusal to honor loss or damage claim. The carrier will furnish you with the necessary forms to file the claim.

Concealed Loss or Damage

When loss or damage is not apparent until the unit is unpacked, a claim for concealed loss or damage is required. When damage is discovered, make a written statement to the carrier for inspection. This request must be made within 48 hours of delivery. Keep all cartons and fillers for inspection. Carrier will furnish inspection report and necessary forms to file concealed loss or damage claim, since such damage or loss is the carrier's responsibility.

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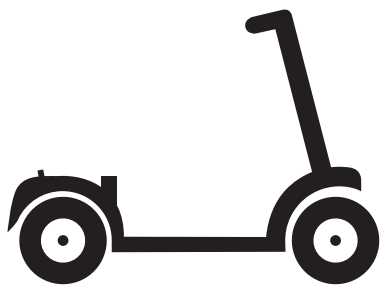
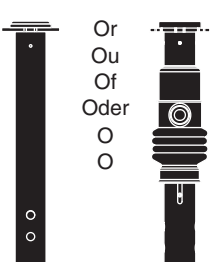
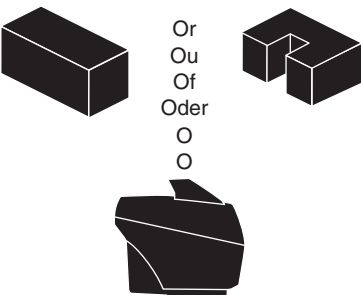
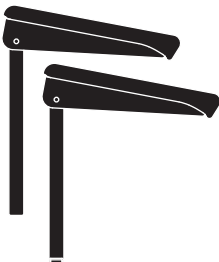
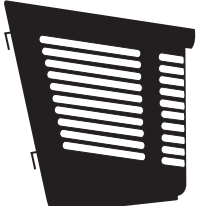
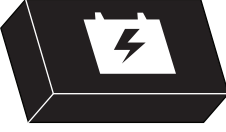
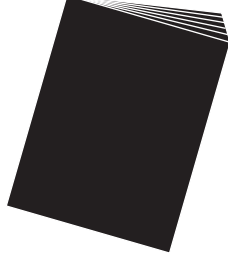
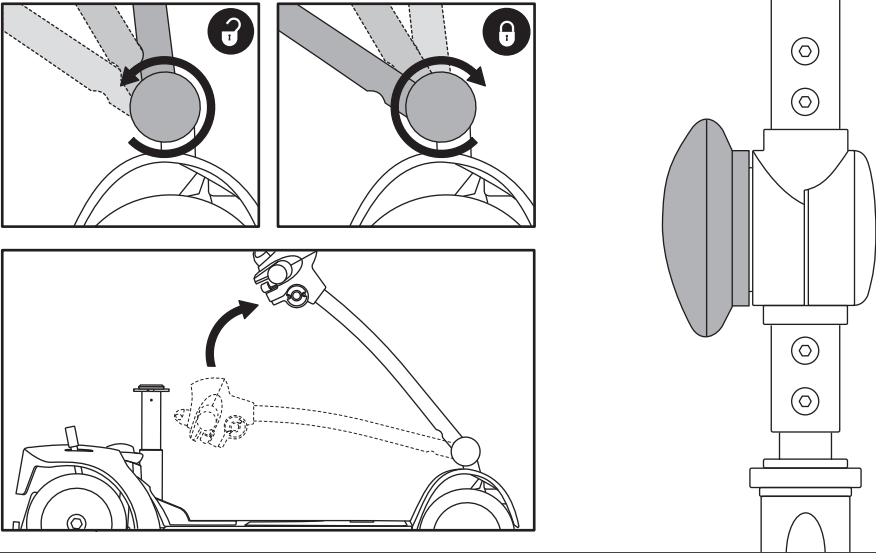
1

Loosen the tiller adjustment knob and raise the tiller, then tighten the adjustment knob.

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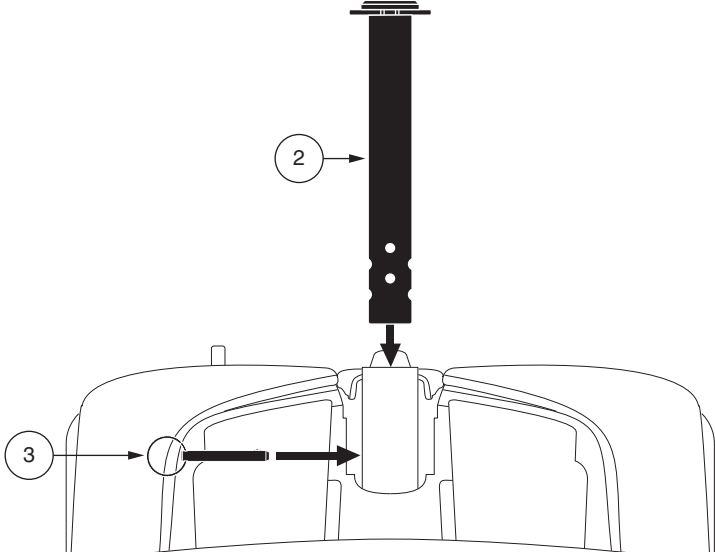
2

Install the seat post and insert the detent pin.

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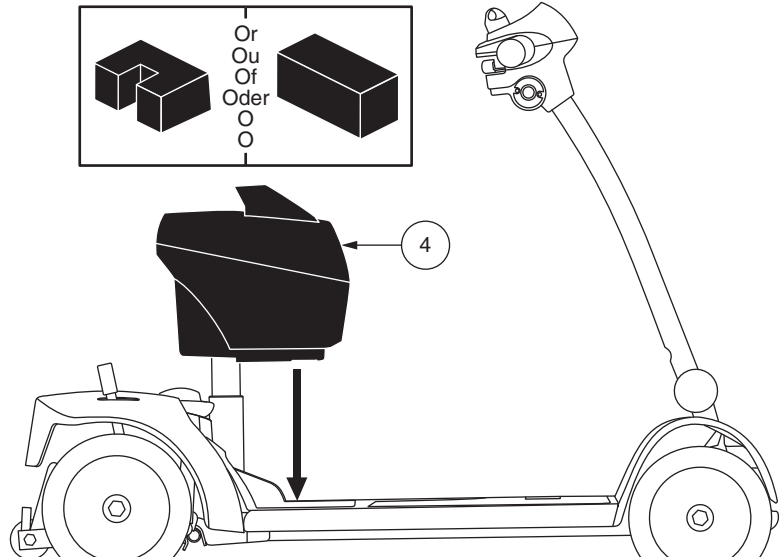
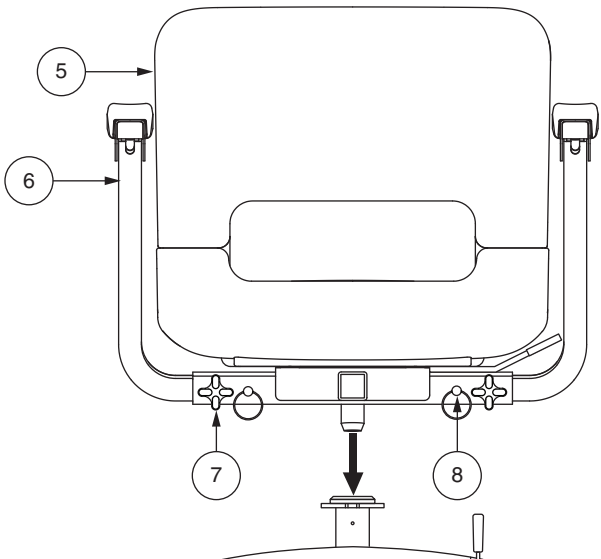
Coloque la caja de baterias.

4

Install the seat and armrests. Install the detent pins (located in the owner's package) and tighten the armrest adjustment knobs.

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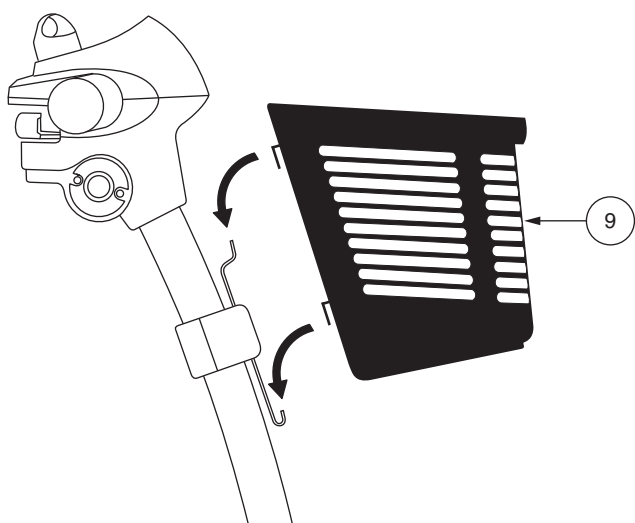
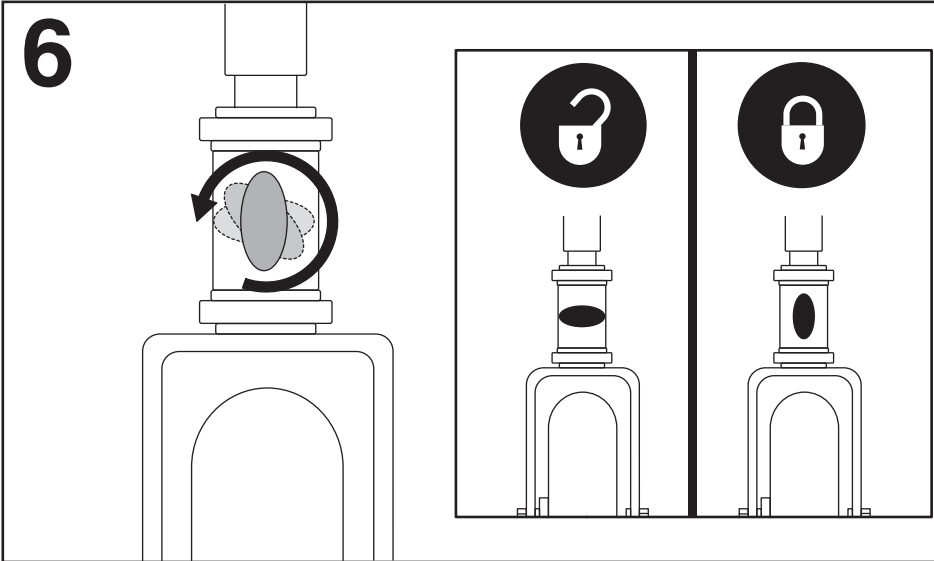
Instale la cesta.

Unlock the tiller by turning the knob as shown.

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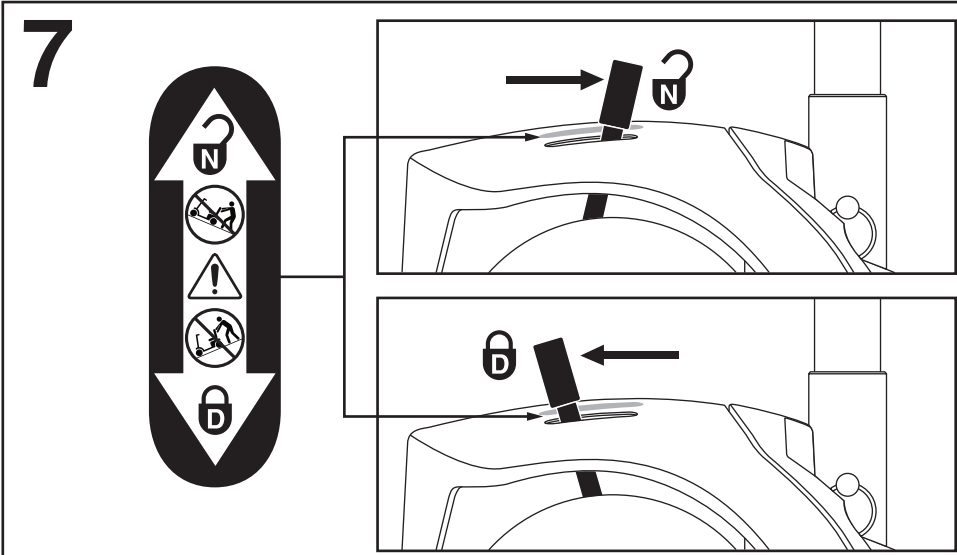
Push the freewheel lever forward to disengage the drive motor (freewheel mode) and position the scooter close to a standard electrical outlet.

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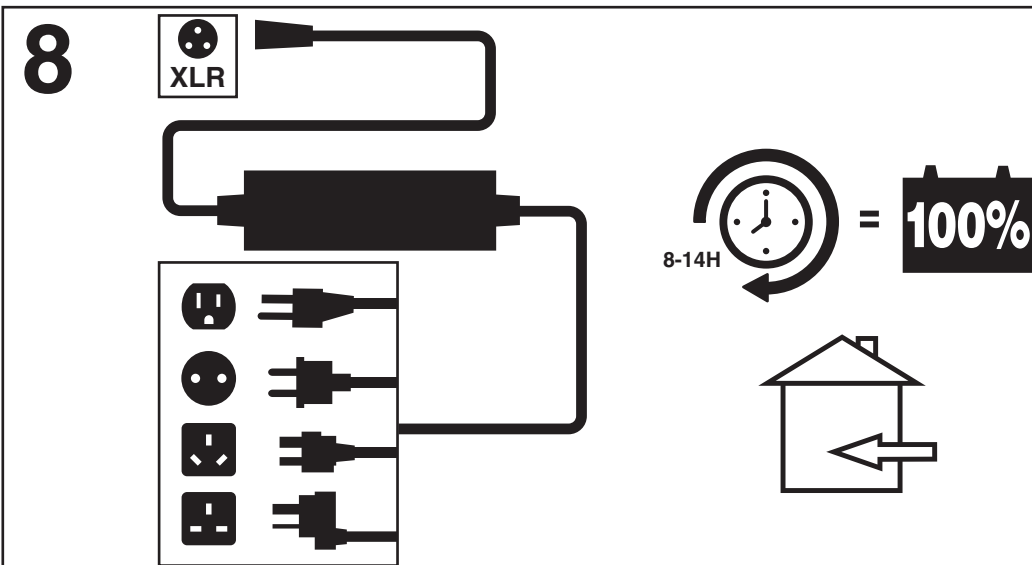
Fully charge the batteries prior to use. Refer to the owner's manual for complete charging instructions.

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This Quick Start Guide offers basic information on initial scooter setup only. Read the owner's manual and all supplemental information thoroughly before operating the scooter for information on safety, battery charging, basic operation, comfort adjustments, disassembly, and care and maintenance.

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